

# Body sculp instrument

Model : BG16

## USER MANUAL



### Cavitation Slimming Machine

- ☞ It Is As Easy To Operate As a Smart Phone
- ☞ The Combination Of Laser Liposuction And Cavitation Technology Offers Visible Slimming Effects.
- ☞ It Can Accelerate The Coagulation Of Cell Tissues And Prevent Muscle Relaxation After Weight Loss
- ☞ It Has Surgical Effects Without The Need For Surgery And Is Highly Accepted By Customers
- ☞ Suitable For All Skin Types And Harmless To The Body

---

---

## Notice

Dear Customer:

Thank you for your choice of the company carefully designed and manufactured instruments, in order to better play the effectiveness of the product, we sincerely recommend:

- Before using the instrument, please read this user manual carefully and keep it for future reference.
- Please install, use and operate properly according to the manual.
- Do not disassemble this unit and modify any parts.
- In addition to our authorized personnel, it is forbidden for others to open the cover and open the host.
- Non-professionals are strictly forbidden to carry out maintenance and renovation without authorization.
- If there is any fault during the operation, please contact us.

**This product does not belong to medical equipment, do not use for disease treatment!**

---

---

## **Functional principle**

### **1. Powerful Lipolysis, Remove Fat**

The use of strong acoustic waves gathered strong acoustic head firing frequency of 40000HZ sound waves into the human body can make the body fat cells have a strong impact and the friction between fat cells movement can effectively consume calories and consume cells of water , So that fat cells shrink, the other when the sound waves can make the cell strong crack impact, the cells burst, fat cells decreased, so as to achieve the effect of removing fat.

### **2. Dissolve Fat, Lymphatic Drainage, Firm Skin, Enhance Skin Elasticity**

With the most advanced RF technology and energy RF in one, you can deep direct fat body, with targeted RF output excellence, so that fat cells in a rapidly active state, the body heat of friction heat, the local temperature Increase, through the sweat glands, enterohepatic circulation and lymphatic excess body fat and toxins excreted, in order to achieve the effect of dissolving fat.

### **3. Improve the orange peel tissue, Slimming shape**

The biological wave generated by the electrode of the instrument stimulates the corresponding acupoints of the body and utilizes different frequencies and pulses to effectively stimulate the body of fat under the interaction of various physical electrons to exercise the body again and thus consume the heat again Fat, in order to achieve the efficacy of plastic body.

### **4. The introduction of magnetic therapy health technology**

Magnetic activation of cells with energy, promote blood circulation; remove free radicals in blood vessels, blood viscosity, lowering blood pressure; sedation, anti-inflammatory, analgesic, swelling; improve immunity; anti-aging effect.

### **5. Anti-aging row acid balance**

According to the acid-base balance, acid-base neutralization theory, a large amount of acidic substances are gathered in the diseased parts of the human body, and the acidic substances are what we call the wastes that are not metabolized by the human body. The 3D stereo carving instrument integrates light, electricity and magnetism , Infrared functions in one, for the modern sub-health conditions, to soothing, physiotherapy, beauty and other functions; through vacuum suction, far infrared, magnetic therapy and optoelectronics, smooth lymphatic channels to speed up the body's metabolism, blood circulation, speed up Toxin metabolism and nutrient absorption, relieve pain, health care, which played the role of row acid acid correction.

---

## Accessories features and operating instructions



Accessory Name	Core Function	Applicable Parts
Ultrasonic Probe (Small) (UI Ultrasound(S))	Promotes metabolism and softens local tissues	Small-area parts such as the face and neck
Ultrasonic Probe (Large) (UI Ultrasound(L))	Accelerates the metabolism of excess substances in the body and assists in fat dissolution	Large-area parts such as the abdomen, thighs, and back
Facial RF Head (Face RF)	Massages facial skin, stimulates collagen regeneration, and improves wrinkles	Face
Body RF Head (Body RF)	Firms the skin and shapes the contours of parts such as the abdomen and buttocks	Abdomen, buttocks, arms
Vacuum RF Head (Vacuum RF)	Combines vacuum adsorption and RF functions to achieve body shaping and simulate cupping effects	Back, abdomen, thighs
Pop-Up Head	Massages the chest and buttocks and assists in lifting the contours of the chest and buttocks	Chest, buttocks
Face Lift Head	Emits microcurrent to enhance the elasticity of facial muscles and improve facial sagging	Face
Body Lift Head	Massages the skin, promotes local metabolism, and assists in shaping the body contour	Abdomen, thighs, arms
Roller (Cift Roller)	Massages muscles, relieves muscle tension, and assists in shaping	Neck, back, and leg muscle groups



---

---

## **Operational recommendations**

① The main role of magnetic therapy in the neck and lumbar health regimen, a site recommended operation time of 15-30 minutes, according to the actual situation of the client (stasis and physical differences) to adjust.

② Before using the probe operation, it is recommended to manually open the need to operate the site for 5-10 minutes of open-hole massage, so that the customer's body has an adaptation process, so as to achieve the most comfortable state.

③ Do physical health projects, with the corresponding function of health oil can be.

④ Parameter adjustment:

Time--According to the actual situation of customers can be adjusted.

Vibration Intensity--Micro-current intensity, adjustable range 0-15。

Heat Intensity--Heat intensity, it is recommended first transferred to 15, preheated for about 5 minutes, and then adjusted according to the actual situation.

Start--start up button。

Pause--Pause button。



--Back to Home page。

## **Treatment tips**

① The probe must be in close contact with the skin during treatment。

② Beautician to adjust the intensity, should communicate well with customers。



③ After the operation, clean the skin with a hot towel。

④ Nursing with a circular exercise or a straight line exercise, do not always stay in the same position, the action should be coherent or pull up。

### **※For the location: body part**

① From the inside of the thigh → thigh → thigh outside the order up to the inguinal direction


② Thigh medial and lateral available Z or large 8-word approach back and forth to promote

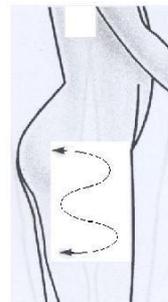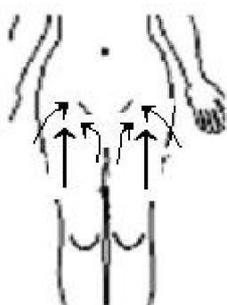

---


③ After all thighs to hip position

④ Lower leg can be used to pull up or use the Z method to push up and down

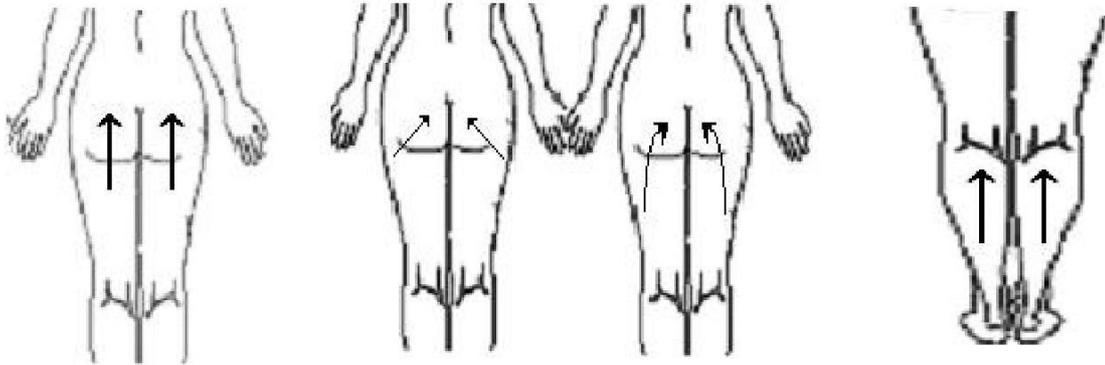


※ **For the location: back and arm**

- ① The back of the spine center point to the axillary lymph node position
- ② After the arm pulled up to the axillary lymph nodes
- ③ The front arm is pulled up to the axillary lymph node position

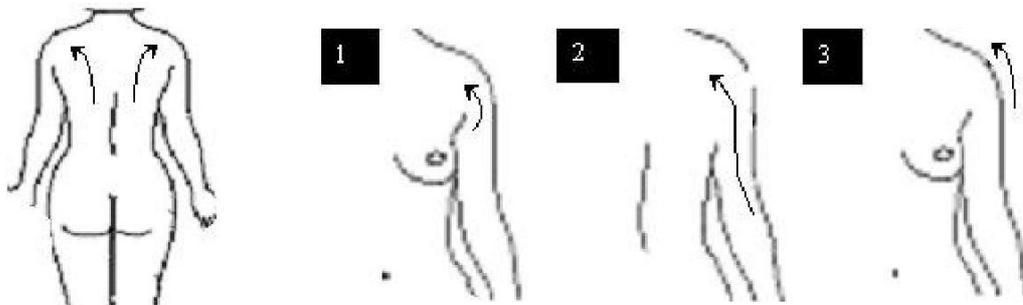


※ **For the location: abdomen**

- ① From the back of the lower spine center began to pull the anterior lobe to the direction of the groin
- ② Clockwise from the upper colon → transverse colon → descending colon sequence to groin position stop
- ③ Stomach belly to A word approach down to the groin direction

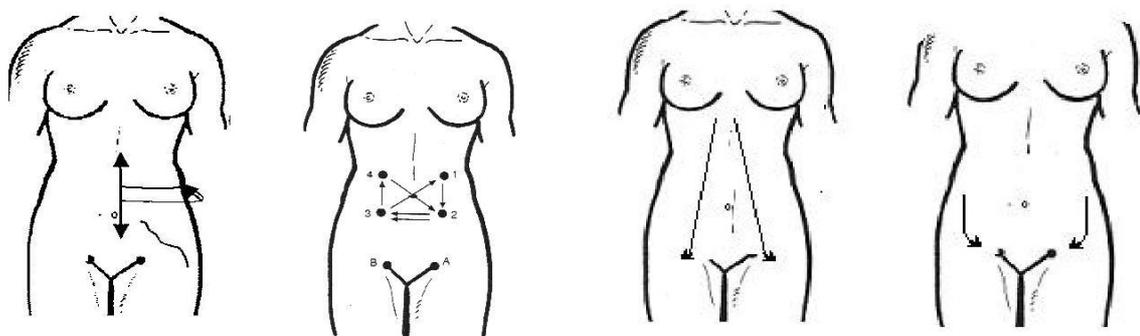


- ① Abdomen: The fluid is drained to the axillary lymph nodes
- ② Lower abdomen: The fluid is drained to the inguinal lymph nodes
- ③ Waist: the liquid drainage to the inguinal shower

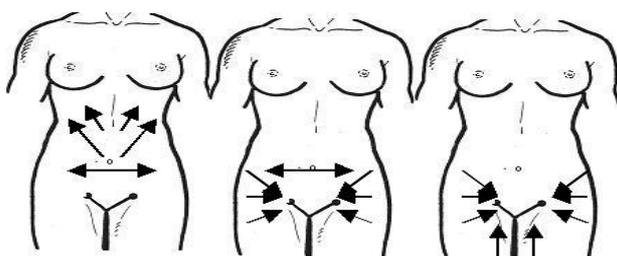

---


※ **For the location: buttocks and legs**

- ① Hind legs: the liquid drainage to the inguinal lymph nodes
- ② Calf: The fluid drainage to the national lymph nodes
- ③ Joint position: joint fluid drainage to the lymph

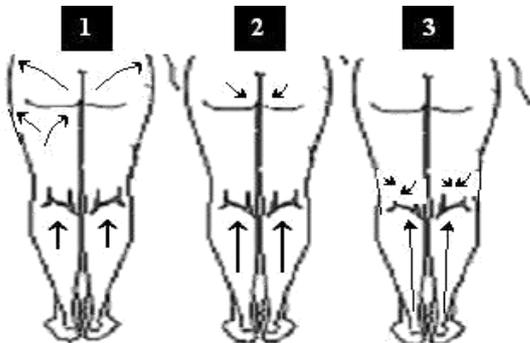

※ **For the location: back**

- ① Shoulder: The liquid is drained to the collarbone lymph nodes
- ② Buttocks: The fluid is drained to the inguinal lymph nodes
- ③ Back: Drain the fluid to the axillary lymph nodes

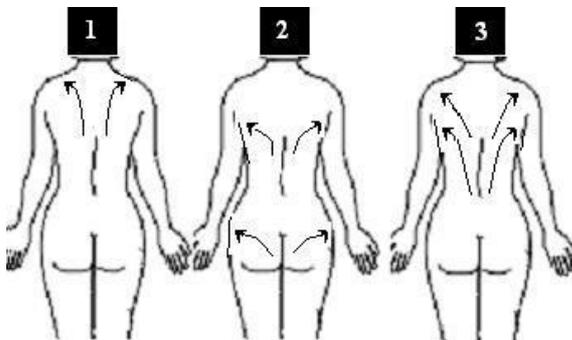

**Handle accessories care**

1. After each operation is completed, the probe should be promptly cleaned, washed with saline, the probe can not be directly into the water cleaning, wipe with a soft rag.
2. Due to the operation of the probe fever, after use, can not be placed directly on the rack, the probe should be cooled and then gently placed on the accessories rack
3. Accessories rack can not be wiped with alcohol, should be used to clean the saline.

## **Precautions**

- ① Instrument is a high-precision electronic equipment, in the custody and place, must not be placed on wet areas.
- ② The instrument uses computer automatic chip operation control, so when in use, try not to use high frequency electrical peripherals, so as not to interfere with the normal operation of the instrument. (Such as electric volume, hair dryer, microwave equipment, etc.). If there is interference, there are generally two phenomena (A. Normal use, suddenly crashes B. LCD display jumped), the general rule of thumb is to turn off the power, restart the instrument.
- ③ Can not be beat, strong collision and so on。
- ④ The mainframe and accessories of the instrument generally need regular cleaning and care, especially not to allow the water to enter the interior of the instrument.
- ⑤ If not for a long time, it is best to use a moisture-proof and dust bag to protect the instrument as a whole, the best power for about 15 days for an hour to avoid the internal electronic components moisture aging.

### **2.Note**

It is prohibited to use under the following circumstances:

- ① Power plug, wire is damaged or hot;
- ② When the jack is loose or shaking;
- ③ Prohibition of damage, processing, excessive bend, twisting plug;
- ④ Do not force the plug pull operation;
- ⑤ Do not use other power cords
- ⑥ Placed in infants, young children can not reach places;
- ⑦ Do not use chemicals to wipe the instrument。
- ⑧ Do not pull the plug with wet hands;

### **4. Disable the crowd**

- ① Serious diseases, such as heart disease, gastric ulcer, severe stomach, duodenal ulcer, hypertension, diabetes and so on.

② Lactating women, abdominal surgery less than three months, menstrual ban.

## **Maintenance And Repair**

1. Each accessory is a sophisticated device, so when in use and the use of the completion, we must pay attention to maintenance, so as to avoid shortening the service life.

2. Accessories to be taken lightly, be sure to pay attention to holding the handle, do not carry the output line is free to put, easily lead to parts within the connector loosening, resulting in poor contact and short circuit.

3. After the use of accessories, do not arbitrarily drop, should be carefully cleaned with a clean solution with clean touch cloth, into the rack. Proposed to use saline or a little detergent for surface cleaning.

4. 40K strong burst fat head 30 minutes if the continuous work time, when the work of 30 minutes, the probe should be allowed to rest for 5 minutes until the surface temperature decreases, and then continue to use, when the accessory is not in use, be sure to remove Accessories, on the corresponding rack, do not let it dry work, so as not to burn the probe.

5. Vacuum negative pressure head: Be sure to pay attention to cleaning, after use is completed, when found in the host of the filter oil or when the product is full, be sure to clean the filter of the product, so as not to affect the suction.

## **Quality assurance**

The instrument from the date of purchase (by bill) host warranty for 12 months, the handle warranty for 3 months, the warranty period of the fault (non-human phenomena) free repair. Exceed the warranty period can also be responsible for maintenance, in accordance with the provisions of the company to collect spare parts and consumables costs (shipping costs borne by the customer). If you need to add or purchase please contact with the company. The following user personal reasons for the failure, does not provide free repair warranty service:

① Unauthorized disassembly, modification of the product caused by the failure; ② Use of non-distribution of the company caused by the failure of hand tools; ③ Use carelessly, beat,

drop caused by the failure;

④ Failure due to lack of reasonable maintenance;

⑤ Not in accordance with the correct use of instructions caused by the failure.

### Warranty Card

product name		Customer Name	
Product model		Contact number	
Date of purchase		contact address	
Repair date	cause of issue	Maintenance content	Repair man